**CE**

Ce produit est conforme au règlement (UE) 2017/745 relatif aux dispositifs médicaux.  
 This product is conform with regulation (EU) 2017/745 on medical devices.  
 Dit product beantwoordt aan Richtlijn (EU) 2017/745 over medische apparatuur.  
 Este producto es conforme con la Directiva (EU) 2017/745 relativa a los productos médicos.  
 Questo prodotto è in conformità con la Direttiva (EU) 2017/745 relativa ai dispositivi medicali.  
 Este produto está conforme à Directiva (EU) 2017/745 relativa aos dispositivos médicos  
 Dieses Erzeugnis entspricht der Richtlinie (EU) 2017/745 über Medizinprodukte.

<b>Référence G / Reference G / Referentie G / Referencia G / Codice G / Referència G / Referenz G / Код G :</b>	134740 134700 134720 134730
<b>Référence P / Reference P / Referentie P / Referencia P / Codice P / Referència P / Referenz P / Код P :</b>	000400 047629 047630 047631 047642

<b>Fonction</b>	Permet de prendre une douche en position assise.
<b>Function</b>	Allows to take a shower in a seated position.
<b>Funcție</b>	Zo kan u zittend een douche nemen.
<b>Función</b>	Permite tomar una ducha en posición sentada.
<b>Funzione</b>	Permette di prendere una doccia in posizione seduta.
<b>Função</b>	Permite tomar um ducho sentado.
<b>Funktion</b>	Ermöglicht es, im Sitzen zu duschen.
<b>Назначение</b>	

<b>Descriptif</b>		<b>kg</b>	150 kg
<b>Description</b>		<b>kg</b>	2,170 kg
<b>Beschrijving</b>			
<b>Descripción</b>			
<b>Descrizione</b>			
<b>Descrição</b>			
<b>Beschreibung</b>			
<b>Описание</b>			

<b>Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnnummer / Номер инструкции</b>	A13470022
<b>Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания</b>	10/10/2002
<b>Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакци</b>	21
<b>Date / Date / Datum / Fecha / Data / Data / Datum / Дата</b>	07/09/2023

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**🇬🇧 GB Do not install this equipment before reading the following instructions:**

**🔧 Fitting instructions.** Before drilling into the wall, physically hold the product (in the lowered position) against the wall so that the stand is PERPENDICULAR to the floor, then mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The 4 fixings points have to be necessarily respected. **🧼 Care and maintenance.** **IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **⚠️ Safety warnings.** The shower seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The shower seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. When the seat is in a raised position, DO NOT USE it as a grab bar. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this shower seat. **🗑️** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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